Rescue SAM: Automated external defibrillator (AED) Defibrillatore esterno automatico (DAE)

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RESCUE SAM is a semi-automatic external defibrillator designed for providing simple and direct voice prompts and indications for a straightforward rescue operation.

It is lightweight and battery powered for maximum portability. RESCUE SAM is designed to treat Ventricular Fibrillation (VF) and Fast Ventricular Tachycardia (VT).

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OPERATING MODE: semi-automatic
WAVEFORM: BTE (biphasic truncated exponential type);
impedance compensated
ENERGY: 200J fixed as new norms AHA 2010
- adult: nominal 200 J into a 50 Ω load
- infant/children: less than 50J using optional attenuated
paediatric pads
SHOCK ENERGY ACCURACY:
According to IEC EN60601-2-4
SHOCK-TO-SHOCK CYCLE TIME: typically less than 20 seconds
PROTOCOL: voice prompts and indicators guide user through
protocol
CHARGE CONTROL: automatic by software (arrhythmia detection
system and charging control)
CHARGE TIME FROM "SHOCK ADVISED":
<8 seconds, typical with new battery
VOICE INSTRUCTIONS: detailed voice messages guide responder
through use of the defibrillator
CONTROLS: shock button, info-button, On/Off button
INDICATORS: LEDs (different colours), info-button
CHARGE COMPLETE INDICATOR:
- Voice prompts (press the shock red button)
- Flashing red shock button
DISARM: once charged, the RESCUE SAM disarms if:
- Patient's heart rhythm changes to non-shakeable rhythm, or
- The SHOCK button is not pressed within 15 seconds after being
armed, or
- The Power ON/OFF button is pressed to turn OFF, or
- The defibrillator pads are removed from the patient or the pads
connector is disconnected
SHOCK DELIVERY: shock is delivered if the SHOCK button is
pressed while the defibrillator is armed
SHOCK DELIVERY VECTOR: via adult defibrillator pads in the
anterior-anterior (Lead II) position or via reduced-energy paediatric
pads in the anterior-posterior position
PATIENT ISOLATION: type BF
BATTERY: 12 Volt DC, 4.2 Ah, lithium manganese dioxide, disposable
long-life
BATTERY CAPACITY: min.200 shocks or 4 hours of operating time
continuous
AUTOMATED AND USER-ACTIVATED SELF-TESTS:
- Daily Self-tests: tests internal circuitry, waveform delivery system,
battery capacity, and software.
- Battery Insertion Test: upon battery insertion, extensive automatic
self-tests and user-interactive test check device readiness.
DATA RECORDING AND TRANSMISSION:
- Optional special battery pack for data transmission
- Data stored: first 60 minutes of ECG and the entire incident's events
and analysis decision.

SIZE: 80x300x270 mm

WEIGHT: 2.2 kg with battery pack installed.

Standard set:

- main unit
- disposable defibrillator pads adult
- disposable battery pack

Optional:

- pre connected defibrillator pads adult/paediatric
- disposable defibrillator pads paediatric
- carrying bag
- wall mounted cabinet
- software with USB cable adapter for PC data
management
- wall mount